

Operating instructions

The wash basin bracket can be adjusted 300 mm / 11 6/8 inches up and down.
Max load is 100 kg / 220 lbs.



Take care to avoid injury to hands when in the vicinity of closing moveable parts.



Take care to avoid injury to your head when in the vicinity of moving mechanical parts.

elite
healthcare

Height adjustment

R4750

The height adjustment is controlled by the lever.

Press and hold the lever up to raise the wash basin, and press and hold the lever down to lower the basin. When the lever is released, the motion stops.

