# QUICK GUIDE Luna IPx4 ceiling lift

#### IMPORTANT

- Make sure that the user's head is at a safe distance from the ceiling lift and strap during transfer.
- The hand control must always be connected in order for the lift to function.

## BATTERY

- 100%
- 50 %.
- MUST BE CHARGED

Charger, key for manual emergency lowering.




### CHARGING



#### ALWAYS REMEMBER

To move the Luna to the charging station and place the hand control into the charger.

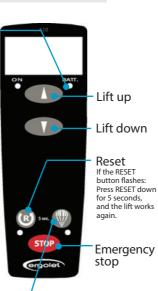

# EMERGENCY LOWERING

1 Activate the powered emergency lowering on the hand control

If the electrical powered emergency lowering is not working, use the manual emergency lowering:

- 2 Take the key from the top of the charger.
- 3 Remove the hand control.
- (4) Remove the plug on the side of the Luna.
- 5 Put the key into the Luna.
- 6 Turn the key counter clockwise to lower the user.









Quick Guide for Luna IPx4 / Version 11 - ENG

Phone: +353 (0)1 8903932 | www.elitehealthcare.ie | info@elitehealthcare.ie